ACTA-EMBRACE™

OPERATION MANUAL

Attaching Compass® 4 Hardware (Quick Release and Fixed, 2 and 4 Point)
to Acta-Embrace™ and Acta-Embrace™ LTS

Compass® 4 Quick Release 2 Point Hardware is shown.

⚠ WARNING: The user SHOULD NOT be seated in the wheelchair until the installation of the back support is complete. Follow instructions in steps 1 through 3 before the user transfers to the wheelchair.

Note: Standard 90° Bracket shown. Installation steps also apply to Pediatric 90° Bracket.

- 1 Loosen the two clamp bolts (A) on both sides using the 5mm hex key. Place and then tighten the clamps onto the wheelchair canes.

Suggestions: Make sure that the clamps are the same height on the wheelchair canes and that they sit square to the wheelchair. Prior to installation, make sure that the cane clamp locking screws (J) are flush against the inner clamp surface.

- 2 Slightly loosen the bolts (B) on both sides using the 5mm hex key. Slide the brackets in and out until the wheelchair canes are relaxed. If further adjustments are needed that the bracket cannot accommodate, follow step 2a for Quick Release or 2b for Fixed Compass:

2a: QUICK RELEASE

Release the toggle (C) and adjust the clamp angles until the wheelchair canes are relaxed. Then lower the toggle to engage the locking mechanism.

2b: FIXED

Loosen (but do not remove) the bolts (D), using a 5mm hex key and an 11mm open-end wrench around cone (E). Adjust the clamp angles until the wheelchair canes are relaxed.

- 3 When the hardware is at the correct position and the wheelchair canes are relaxed, securely tighten the bolts (B) on both sides. If the bolts (D) have been loosened, they should be tightened to between 90-110 in-lbs., using a 5mm hex key and placing an 11mm open-end wrench around the cone (E).

⚠ IMPORTANT: The user may transfer to the wheelchair ONLY after the bolts have been securely tightened as described in Step 3.

- 4 **Height Adjustment:** If height adjustment is needed, slightly loosen bolts (A) or (B) on both sides using the 5mm hex key. The back support can slide up and down using either the slot on the back support shell or on the wheelchair canes. Tighten all four bolts (A) or (B) at the desired height.
- 5 **Depth Adjustment:** If depth adjustment is needed, loosen bolts (F) on both sides, using the 5mm hex key. The back support can slide forward or backward up to 2 inches total. Tighten both bolts (F) at the desired depth.

Suggestion: If additional depth adjustment is needed, Compass can be used in alternate configurations. See the example on the last page.

- 6 **Recline Adjustment:** Using the 5mm hex key, loosen the top and bottom bolts (G) on both sides until the back support can be tilted. Adjust, then tighten all four bolts (G).

⚠ IMPORTANT: When installation and adjustments are complete, all bolts should be tightened to a minimum of 85 in-lbs.

⚠ WARNING: The locking screws (H) and (J) should be tightened only until contact is made. Over-tightening of the locking screws (H) may cause unwanted changes in the recline position.

- 7 Tighten the locking screws (H) on both sides using the supplied 2mm hex key to lock angle rotation. These make contact with recline bolts (G) and prevent angle rotation but may also be used for minor angle adjustments if bolts (G) are loose.
- 8 **Locking the clamp:** To make contact with the wheelchair canes and lock the clamp into its final position, tighten the locking screw (J) on both sides, using the 3mm hex key.

PARTS INCLUDED

All COMPASS® 4 Styles:

Complete Acta-Embrace or Acta-Embrace LTS Assembly with mounting hardware
5mm hex key, 3mm hex key, 2mm hex key


4 Point Hardware:

90° Assembly (includes additional straps for attachment to larger diameter wheelchair canes.)
½" hex key, 4mm hex key

PARTS NEEDED

11mm open-end wrench

*(C) relates to Quick Release Hardware Only
(D) & (E) relate to Fixed Hardware Only
(K-R) relate to 4 Point Hardware Only*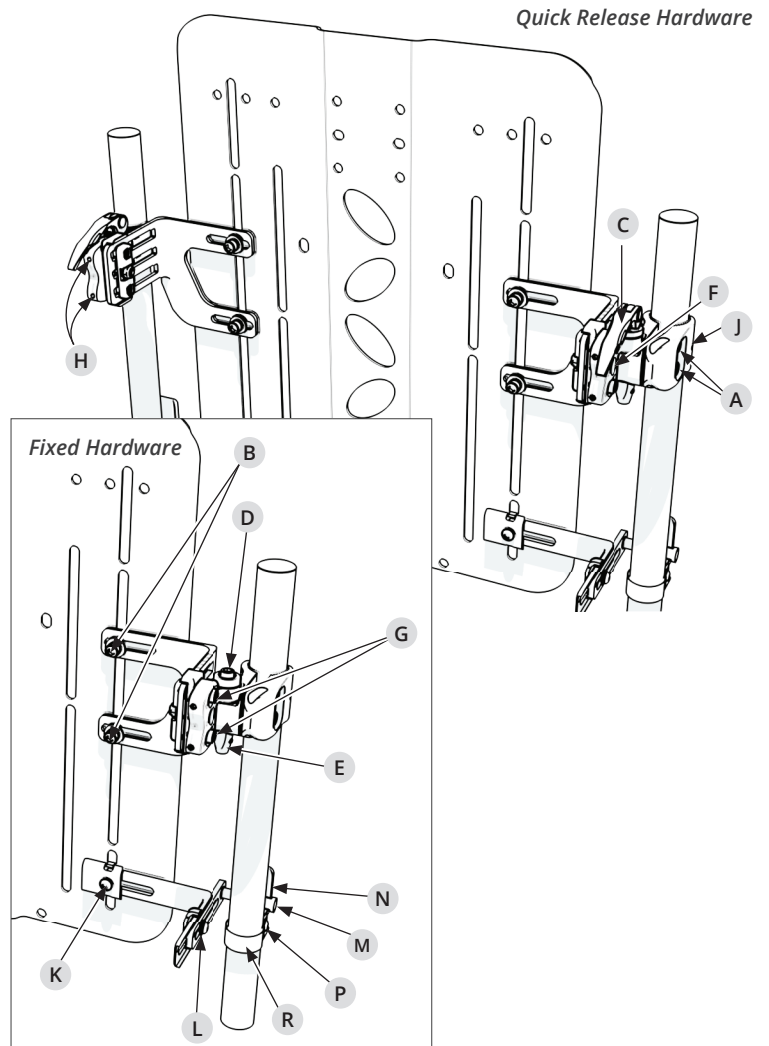


Compass® Hardware (4 Point)

Note: If an alternate Compass 2 point configuration was used above, Compass 4 Point hardware may need to be reconfigured.

- 1** Adjust the mounting brackets: Slightly loosen the bolts (K) and (L) on both sides, using the 4mm hex key. Adjust the hardware so that mounting pegs (M) are resting against the wheelchair canes, as shown. Securely tighten the bolts (K) and (L).
- 2** Attach the cane mounts (N) to secure each mounting peg against the wheelchair cane: Remove the screw (P) using $\frac{1}{8}$ " hex key to release one side of the cane strap (R).
Wheelchair Cane Size and Straps: Use the pre-attached short strap as is for $\frac{7}{8}$ " and 1" canes. Use the inner strap holes on the shorter strap for $\frac{3}{4}$ " canes. Replace the strap with the longer one that is provided in the kit for $1\frac{1}{8}$ " and $1\frac{1}{4}$ " diameter canes.
- 3** Place the cane mounts (N) just below the mounting pegs (M) and wrap the strap (R) around the wheelchair cane. Replace the screw (P) and tighten. Complete for both left and right sides.
- 4** To fully secure the strap: Tighten the screws located on the front of the cane mounts (not shown).

Note: See the last page for using moldable aluminum stays on Acta-Embrace back supports.

(C) relates to Quick Release Hardware Only
(D) & (E) relate to Fixed Hardware Only
(K-R) relate to 4 Point Hardware Only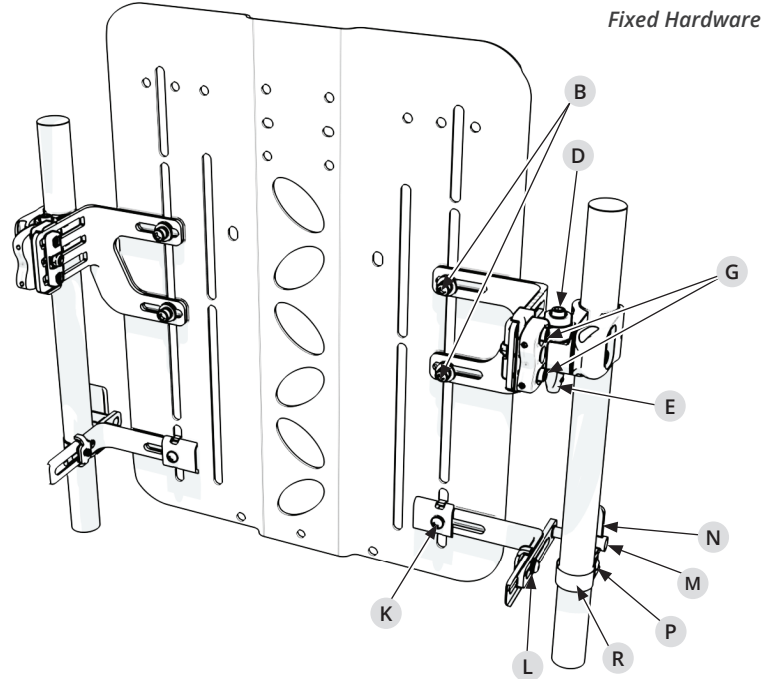


Acta-Embrace supports have aluminum stays inserted in the cover, which can be molded to meet the contour of the spine.

How to mold the aluminum stays:

- 1** To identify gaps or areas that require positioning adjustment, feel between the back support and the user's back.
- Suggestion: To accommodate for rotation without having to rotate the entire back support, adjust the placement of the aluminum stays.*
- 2** Unzip the top of the back support cover, and remove the aluminum stays.
 - 3** Mold the aluminum stays using your hands and a firm surface to fill in any gaps and provide positioning.
 - 4** Reinstall the aluminum stays, and then have the user rest against the back support.
 - 5** Repeat the process, making adjustments until the proper support and positioning goals have been met.
 - 6** Close the cover.
- Suggestion: Aluminum stays can be re-molded for changes in condition and positioning needs over time.*

Alternate Compass 4 Configurations

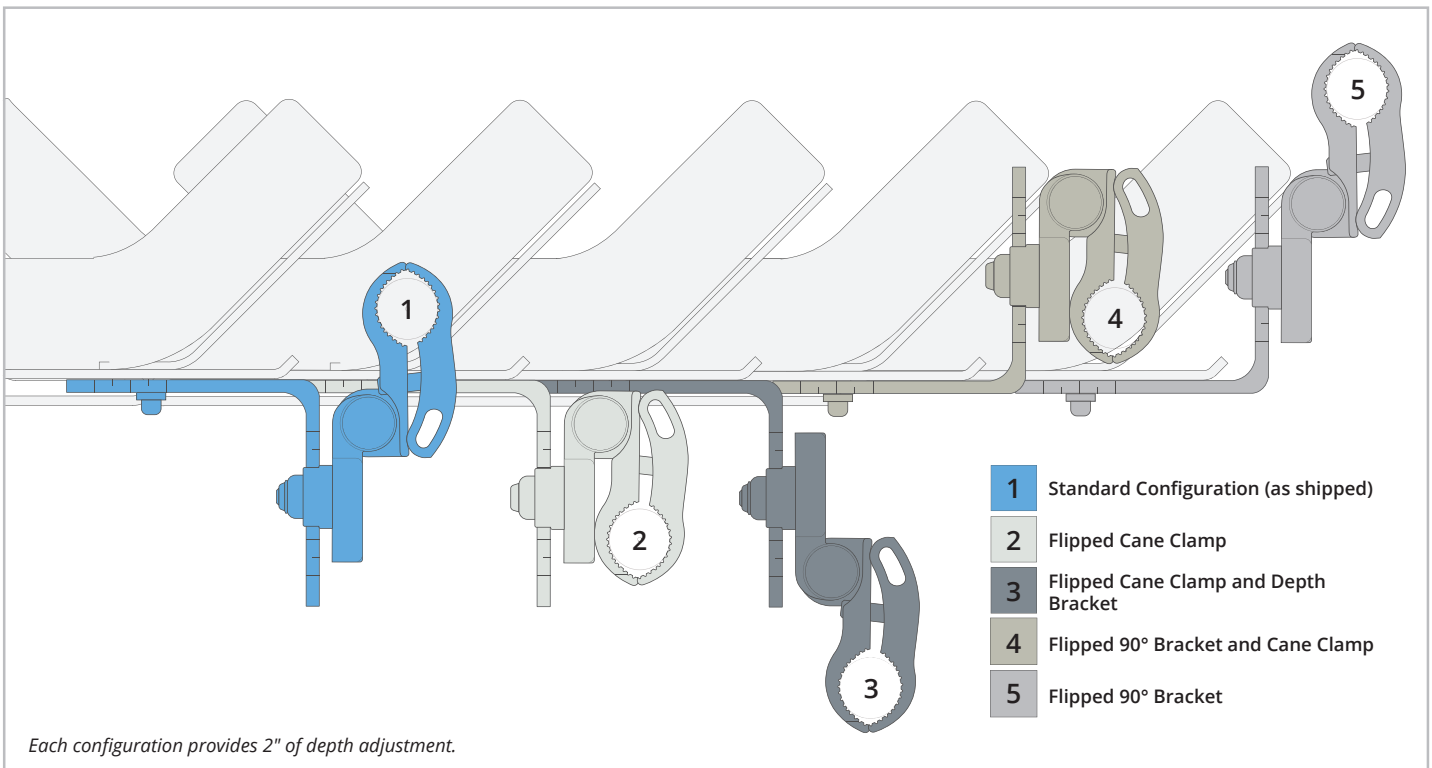Contact Information: Phone: 1.800.736.0925 Fax: 406.522.8563 **Customer Support:** orders.comfort@permobil.com

Manufactured by The Comfort Companies, LLC, New Berlin, Wisconsin

Permobil® is a registered trademark of Permobil, Inc.

Compass® and Acta-Embrace®, are registered trademarks of Permobil.

The Comfort Companies, LLC is a part of Permobil Seating and Positioning.

Note: "We," "Us" and "Our" refers to Permobil. "You" refers to the product purchaser or user.

Product information is changed as needed; current product information is available at permobilus.com.

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© 2018 Permobil
Fabriqué par The Comfort Companies, LLC, New Berlin, Wisconsin
Permobil® est une marque de commerce enregistrée de Permobil, Inc.
The Comfort Companies, LLC fait partie de Permobil Seating and Positioning.