(en) Urinary Night Drainage Bag - STERILE Fluid Pathway

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Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA

Hollister ULC
Foxford Road, Ballina
Co. Mayo, Ireland

Manufactured in India by:
Hollister Medical India Private Limited
Neutral Code # HR-676-B(H)

Hollister®

©2021 Hollister Incorporated.
913153-1121

(en) Urinary Night Drainage Bag

Contents

- Urinary night drainage bag with attached tubing with sterile fluid pathway
- Built-in hanger fixation

Product Description:

Urinary night drainage bag with drain tap, extension tubing and built-in hanger fixation. Designed to be emptied. Extension tubing has a sample port for urine sampling. The extension tubing diverts urine from a sheath (catheter) to the urinary night drainage bag. Built-in hanger fixation supports the urinary night drainage bag on a stand or bed frame.

Product is made of Polyvinylchloride (PVC) and polypropylene.

Indications and Intended Use:

A urinary night drainage bag collects and contains urine for individuals with urinary incontinence.

WARNINGS


To help reduce the potential for infection and/or other complications, do not reuse. Reprocessing, cleaning, disinfection, and sterilization may compromise the product. This may result in odor or leakage. The fluid pathway is only sterile with protective caps intact and in place.

Notes:


Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

INSTRUCTIONS FOR USE:


Applying the urinary night drainage bag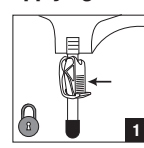

Ensure the drain tap is closed.

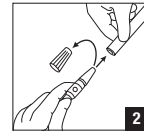

Remove cap from stepped mount connector if present. Connect the stepped mount connector to the sheath (catheter). Make sure tubing is not kinked or obstructed.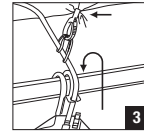







Place the built-in hanger fixation on stand or bed frame. Tubing may be supported using the bedsheet clip.

Please keep the bag off the floor.

Drawing a urine specimen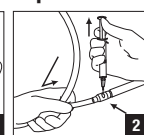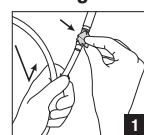

Draining the urinary night drainage bag

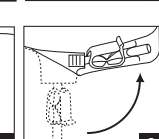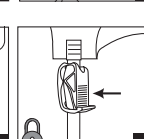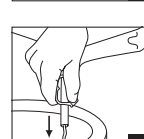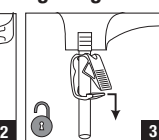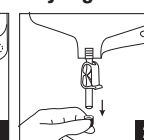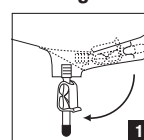To remove and dispose (Please follow all local disposal regulations)

Number of Units

MD Medical Device (as applicable).

Not made with natural rubber latex.
CE mark as required

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