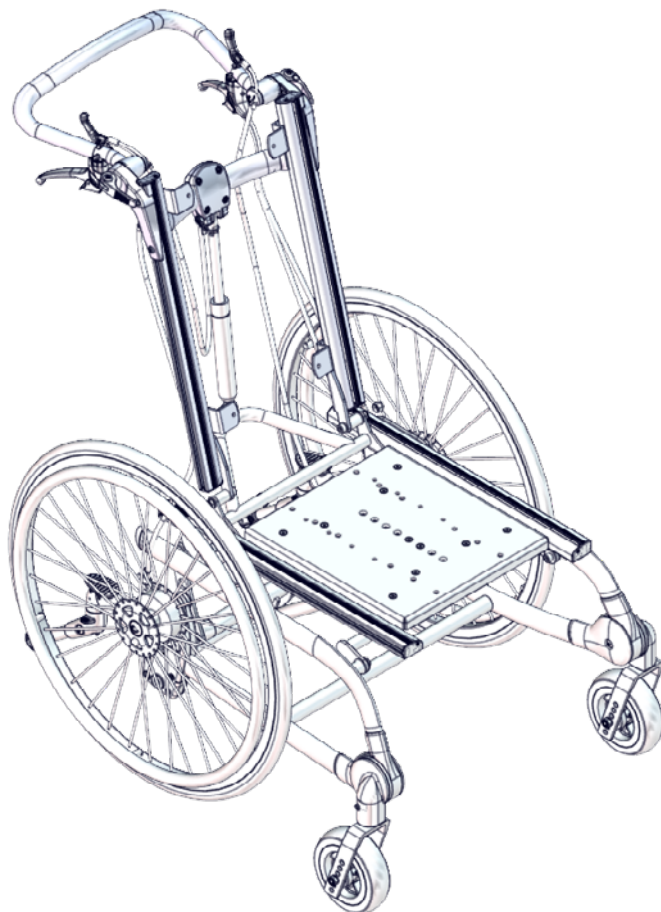
# User manual

Combi Frame:x

**R82**<sup>®</sup>  
by Etac


M1082 - 2023-12-05 - Rev. 13

[www.etac.com](http://www.etac.com)







	3		26
	4-5		27
	6-7	mm (inch) kg (lb)	28
	8		29-30
	9		31-41
	10-11	ENGLISH DEUTSCH	42-46 47-53
	12	NEDERLANDS DANSK	54-60 61-65
	13	NORSK SVENSKA	66-70 71-75
	14-15	SUOMI ESPAÑOL	76-80 81-87
	16-17	FRANÇAIS ITALIANO	88-94 95-100
	18-19	PORTUGUÊS PORTUGUÊS DO BRASIL	101-107 108-112
	20	РОССИЯ POLSKI	113-119 120-125
	21	ČESKY 中國	126-130 131-135
	22	日本 ΕΛΛΗΝΙΚΗ	136-142 143-149
	23	ÍSLENSKA	150-154
	24		
	25		

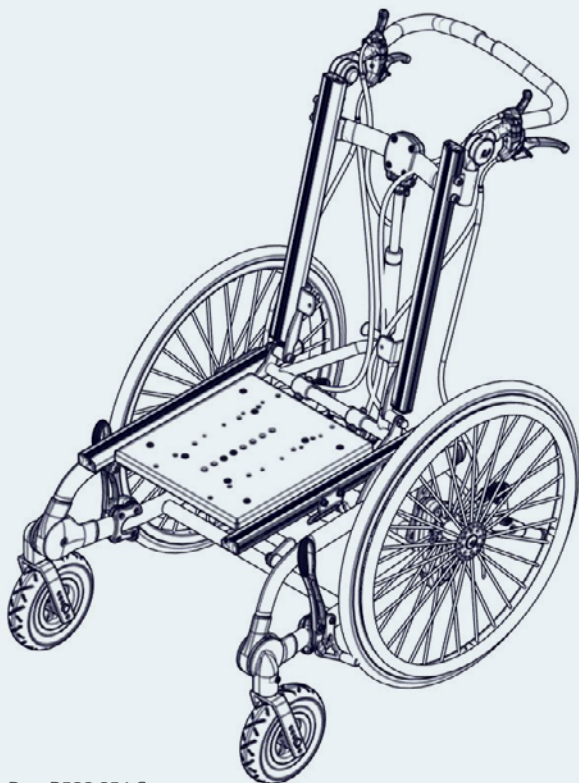U.S. Patent Des. D588,051 5  
Norwegian Design Reg. No. 80196

### Combi Frame:x

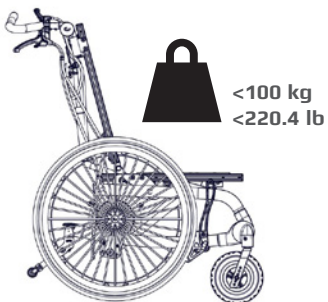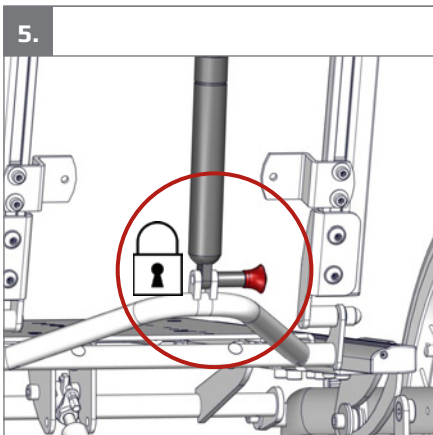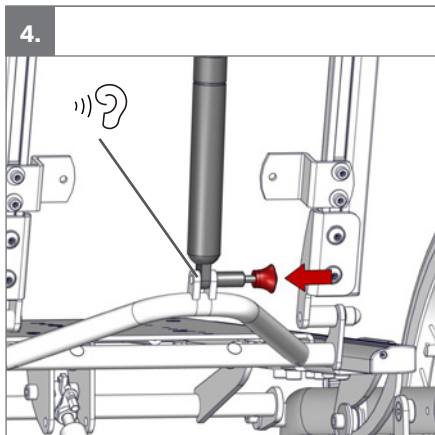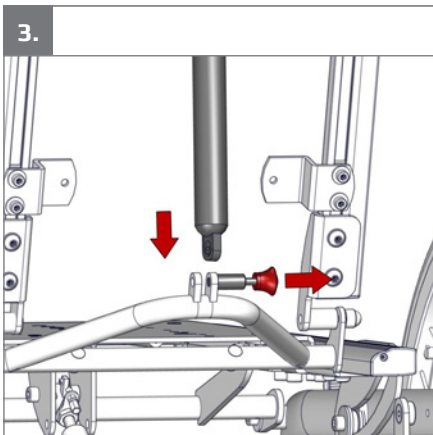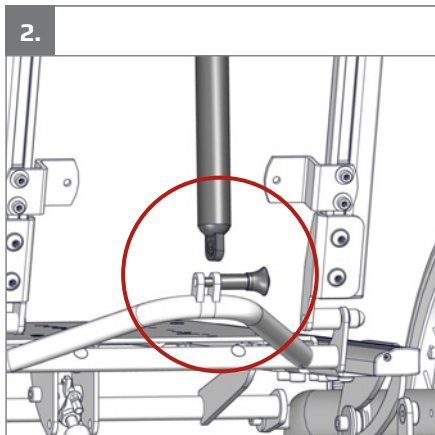
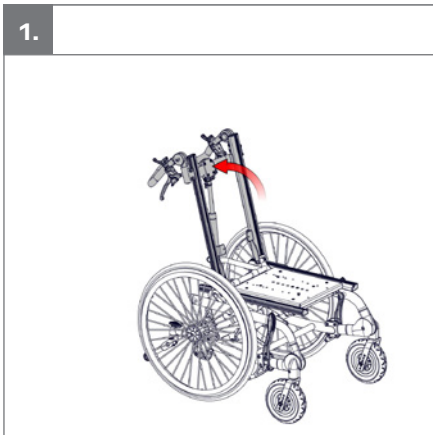
6.

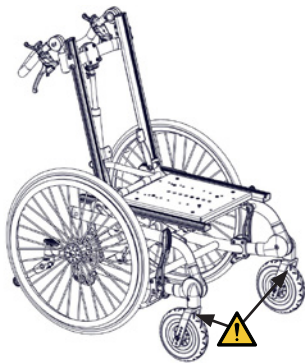
01 

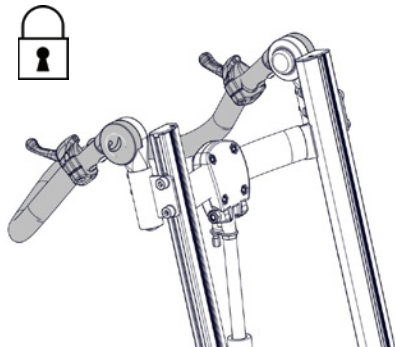
7.

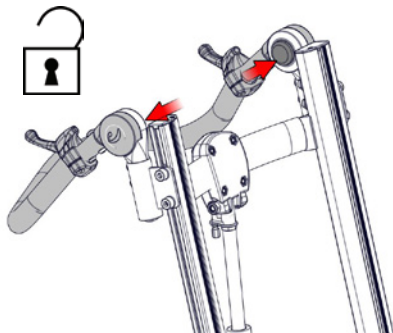
8.

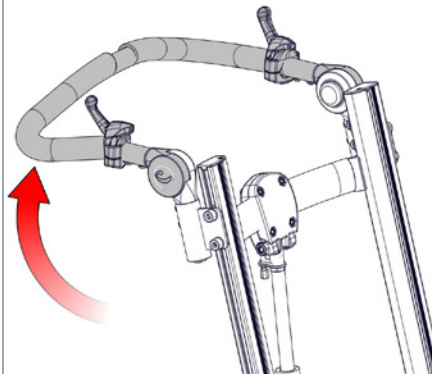
9.

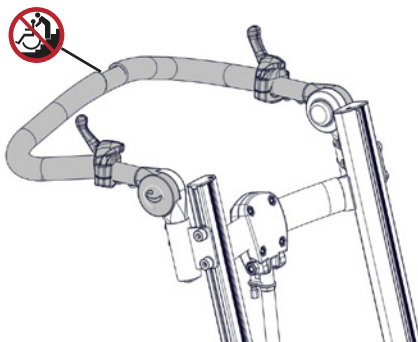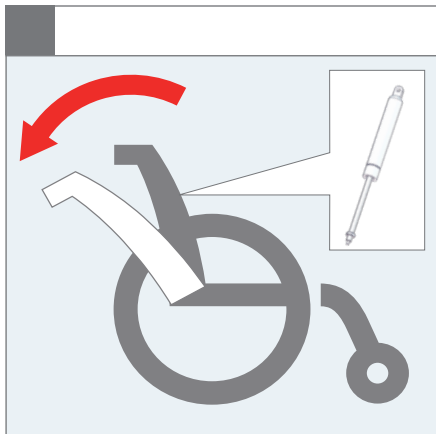
10.







1.





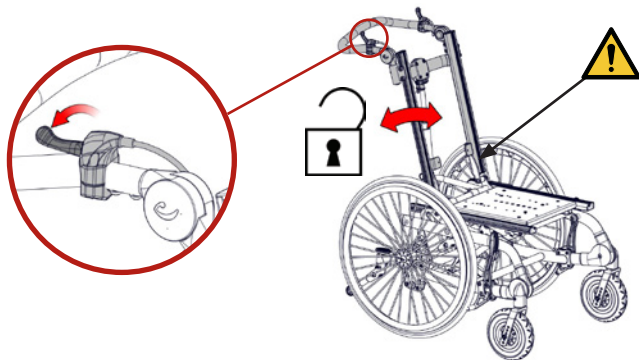
05

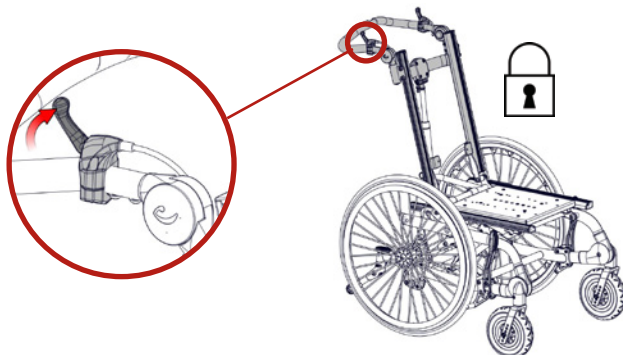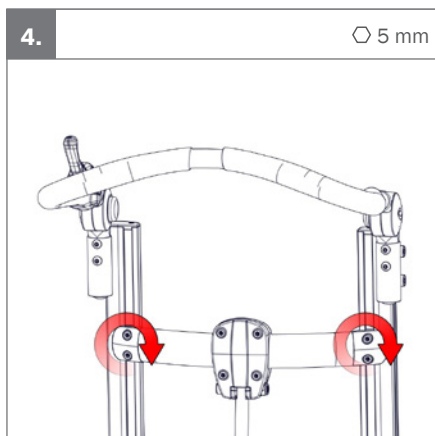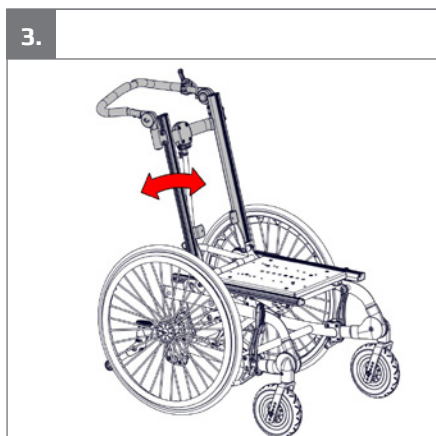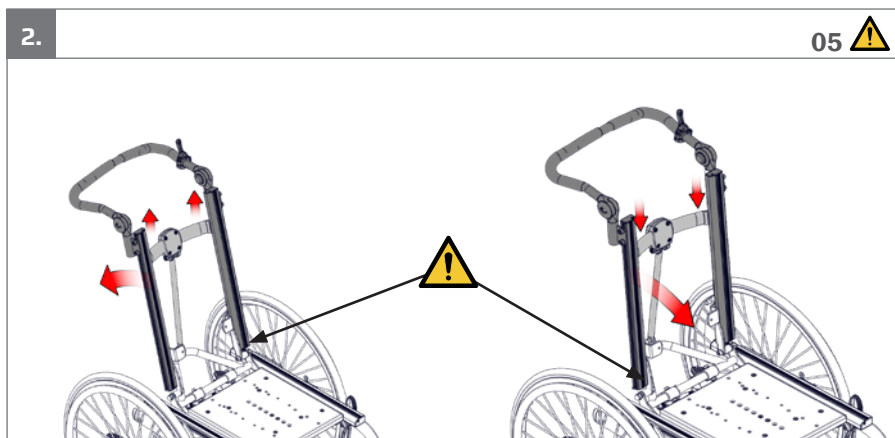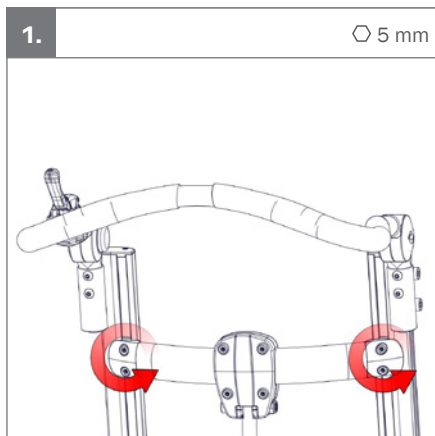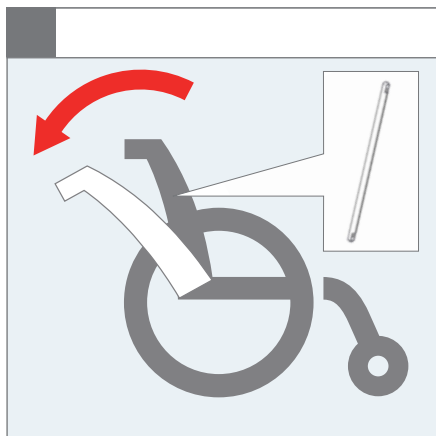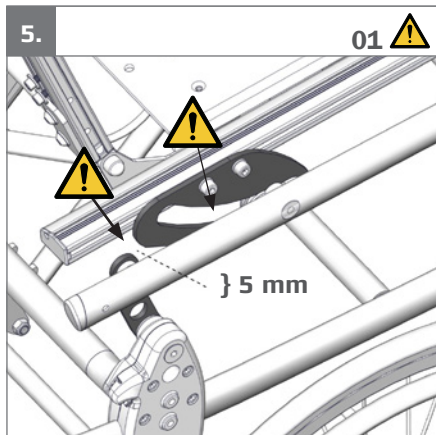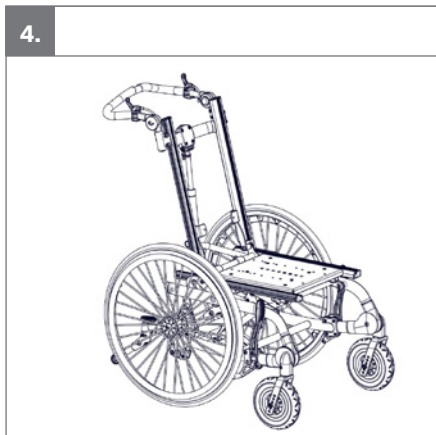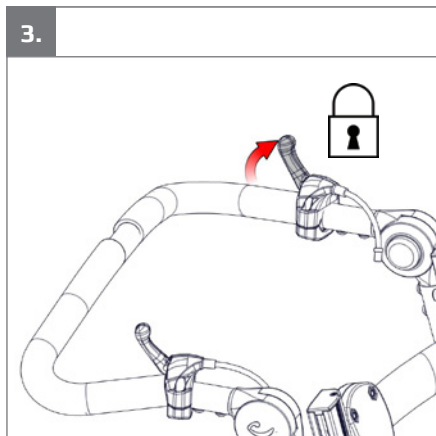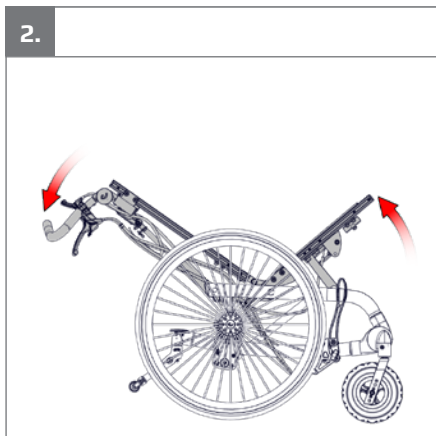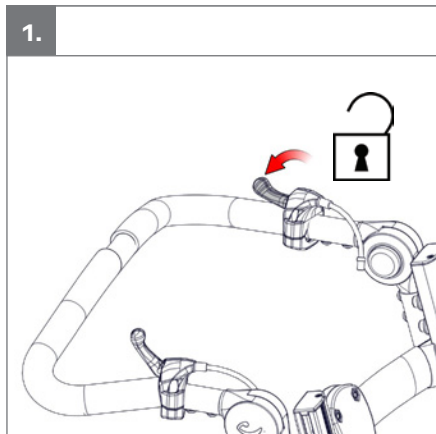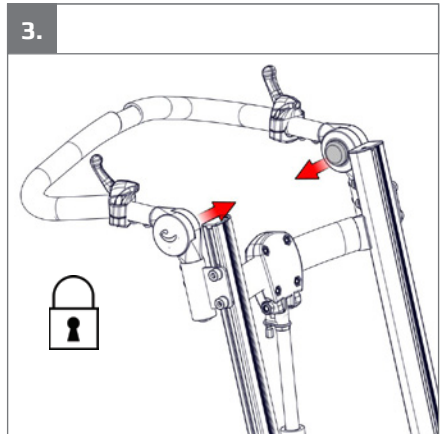
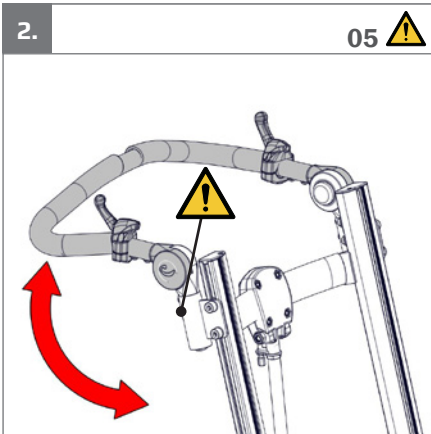
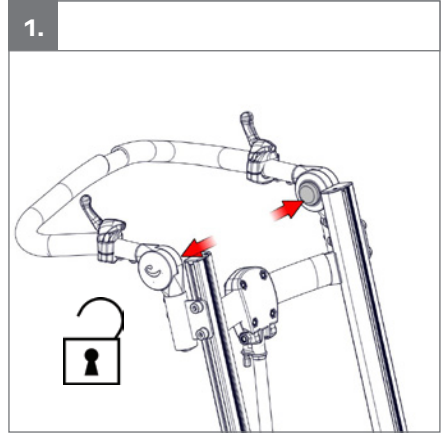
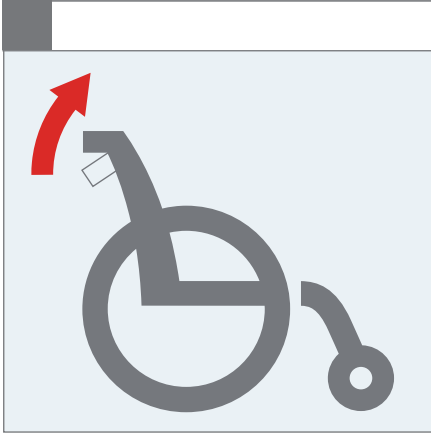
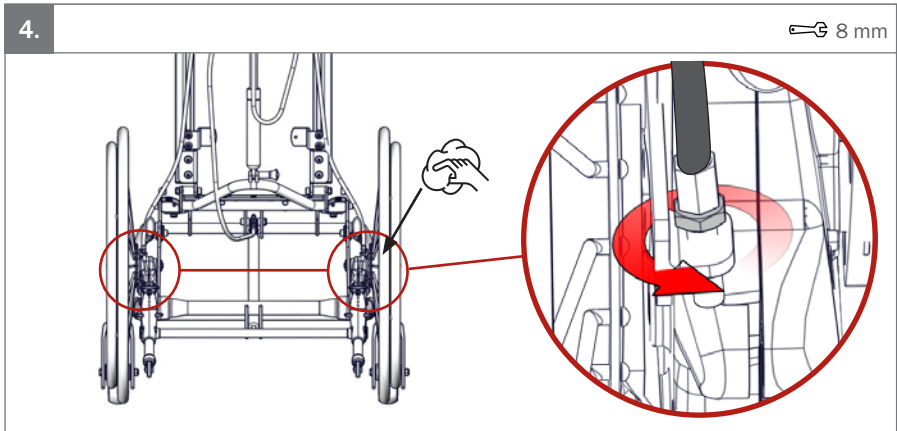
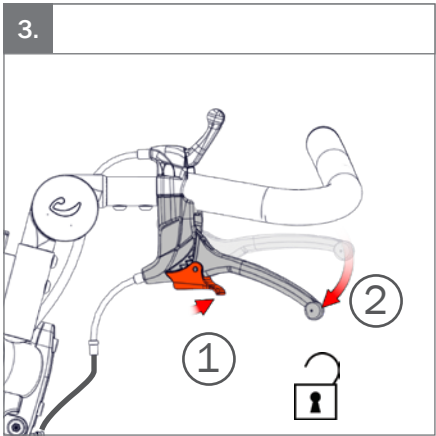
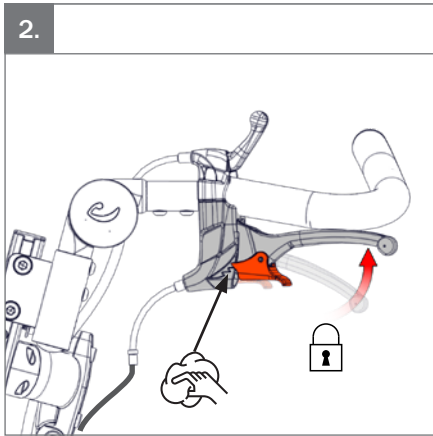
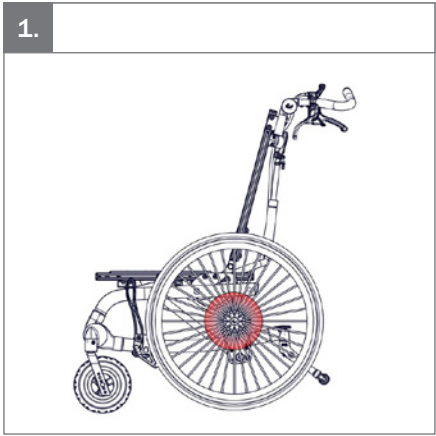
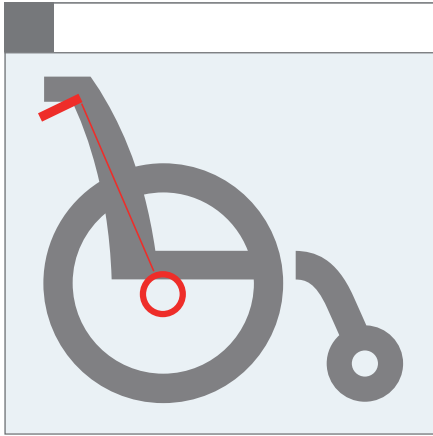
2.


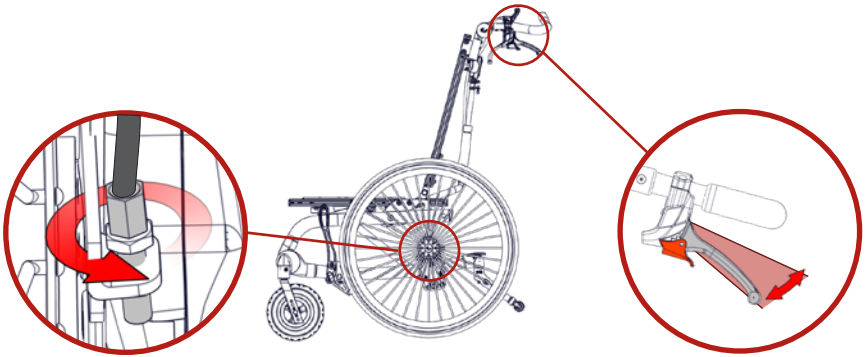
5.

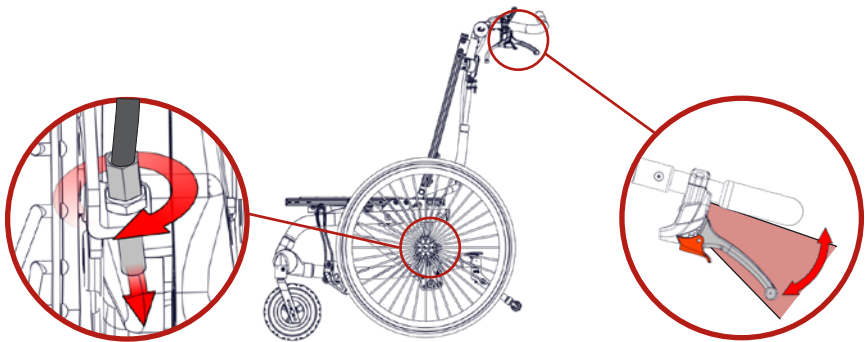
6.






7.

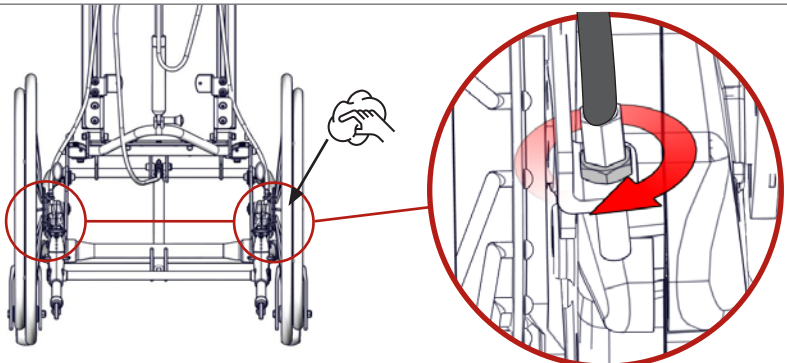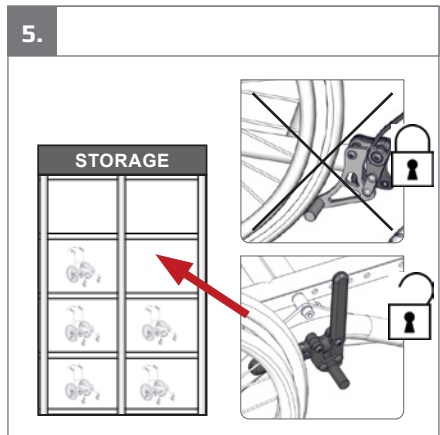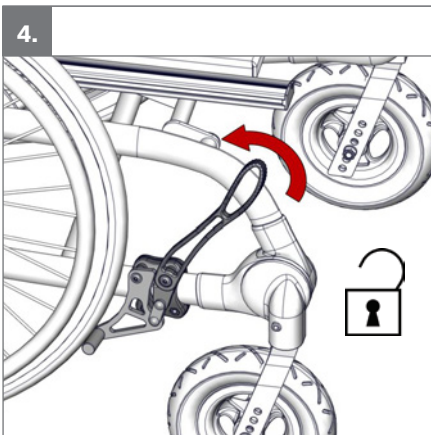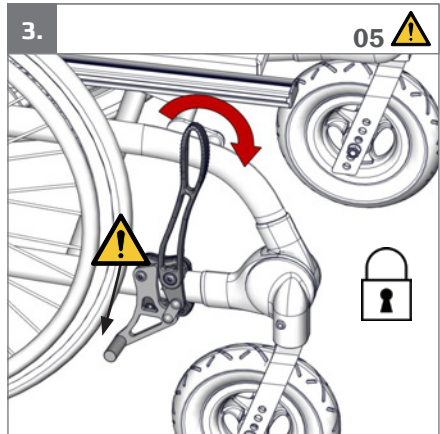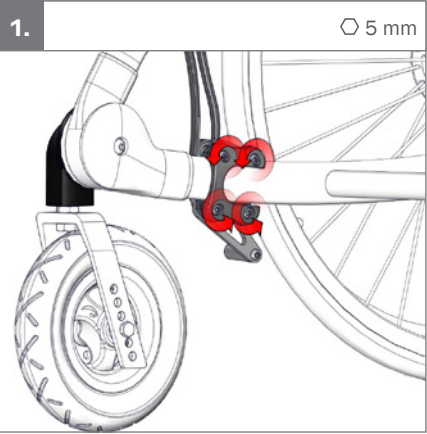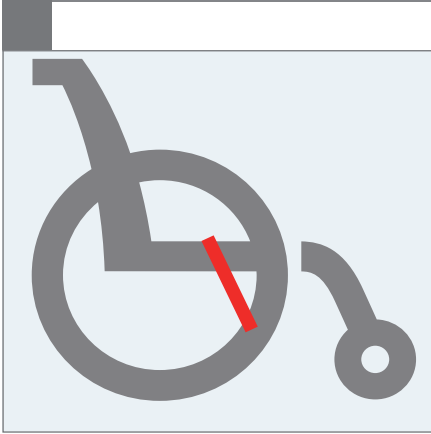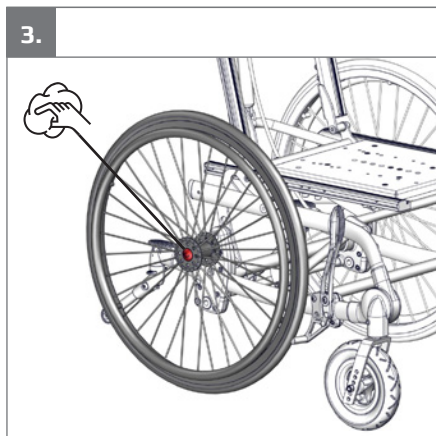
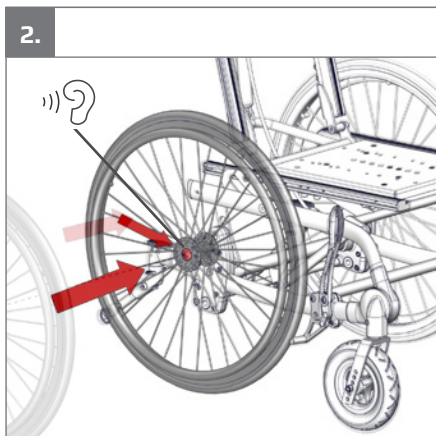
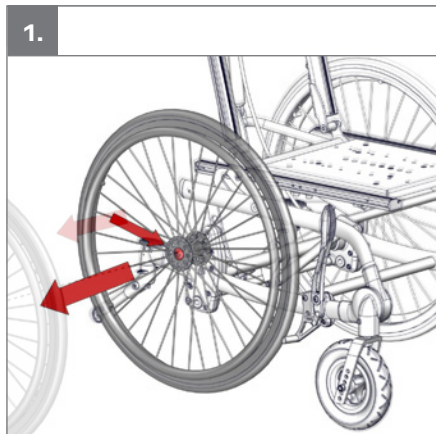
 8 mm



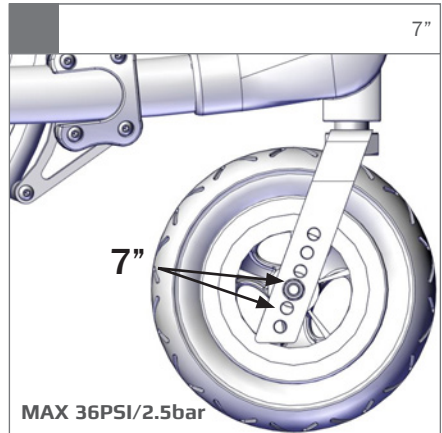
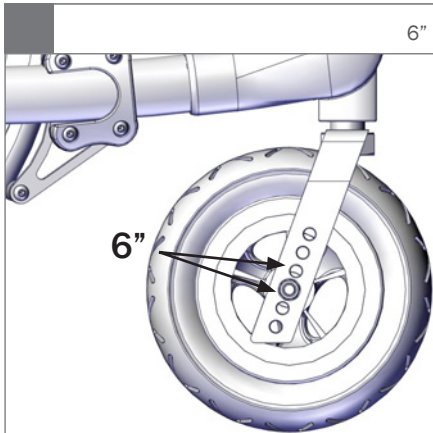
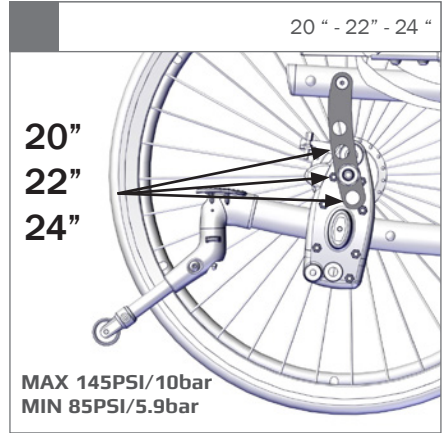
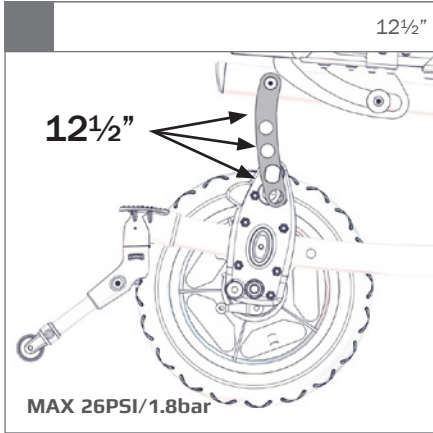
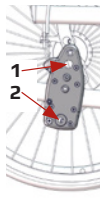
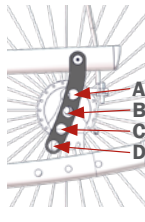
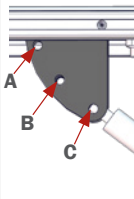
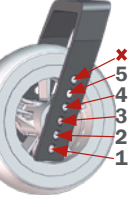
Rear wheel	Axle position	Front wheel	Height adjustment	Seat height	Gas spring position	Front wheel position			
						6"	7"		
	12½"	2	6"/7"	A	430	B	4	N/A	
				B	460	B	2	3	
				C	490	C	1	2	
	20"	1	6"	C	430	A	4	N/A	
				C	460	B	3	N/A	
				D	475	C	1	N/A	
	22"	1	6"/7"	B	430	A	4	N/A	
				C	460	B	1	3	
				D	490	C	1	3	
	24"	1	6"/7"	B	430	B	5	N/A	
				B	460	B	1	3	
				C	490	C	1	3	
									
			12½"	20"	22"	24"			
Handrim diameter			N/A	445 mm (17½")	480 mm (19")	530 mm (20¾")			

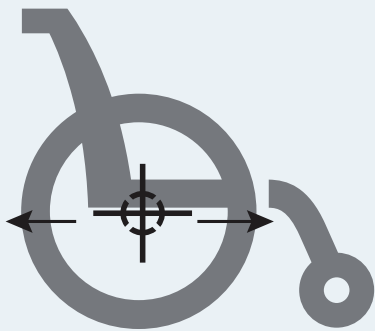
02

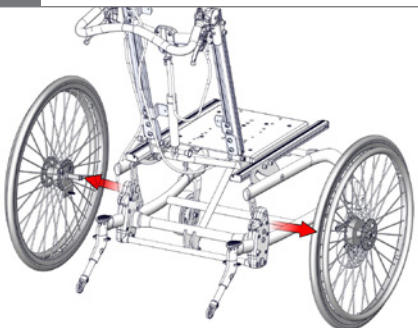1.  $\odot$  4 mm & 10 mm

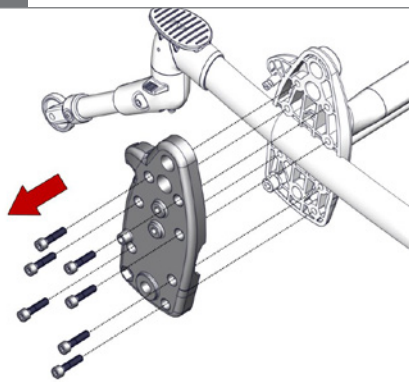
2.


3.

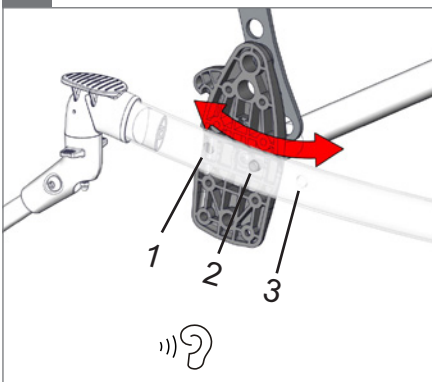
$\odot$  5 mm

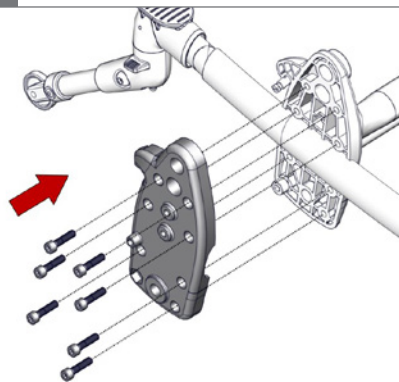4.

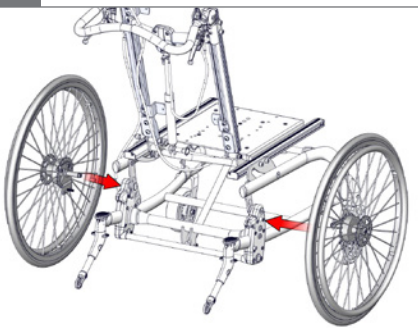
5.








6.

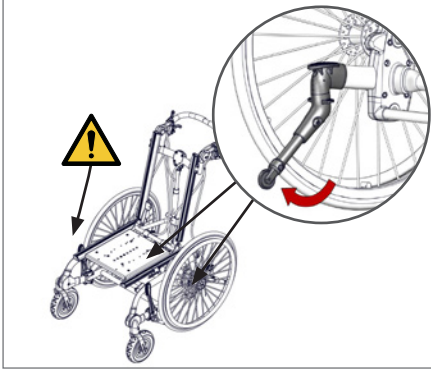02

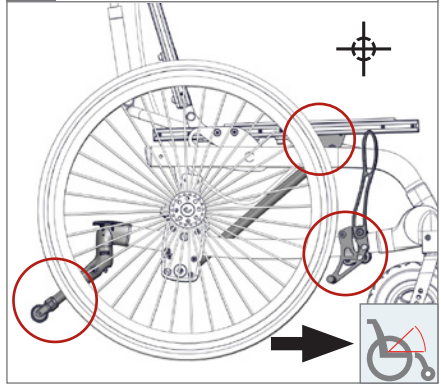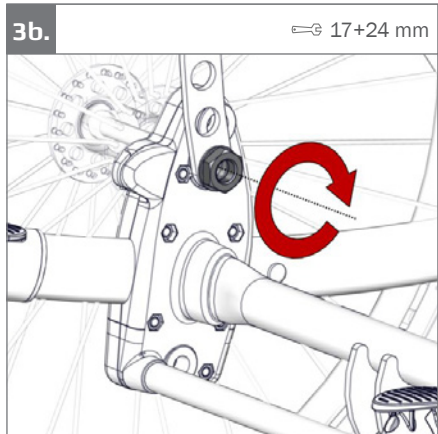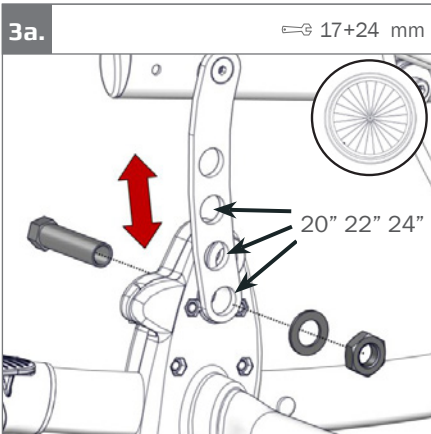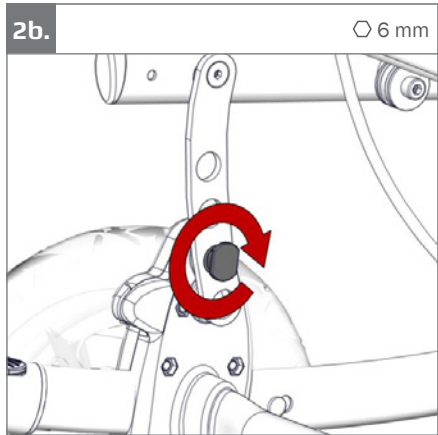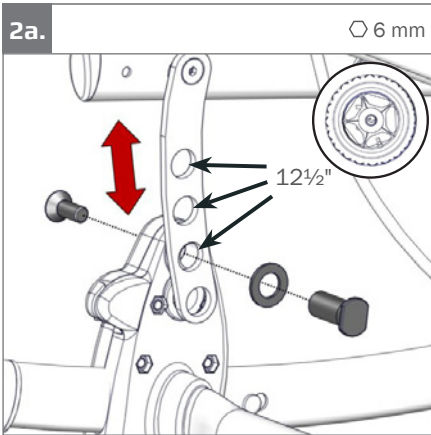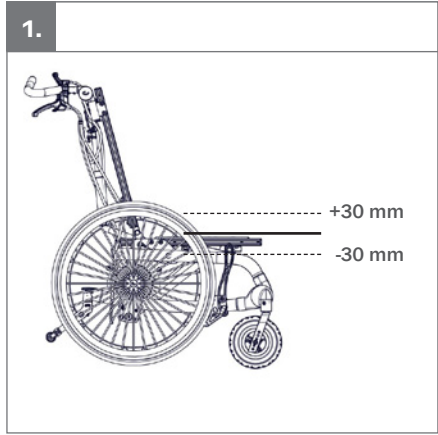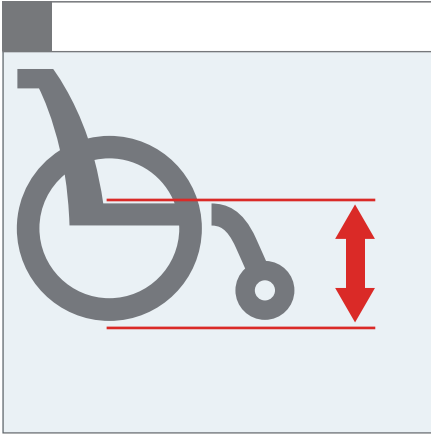
7.

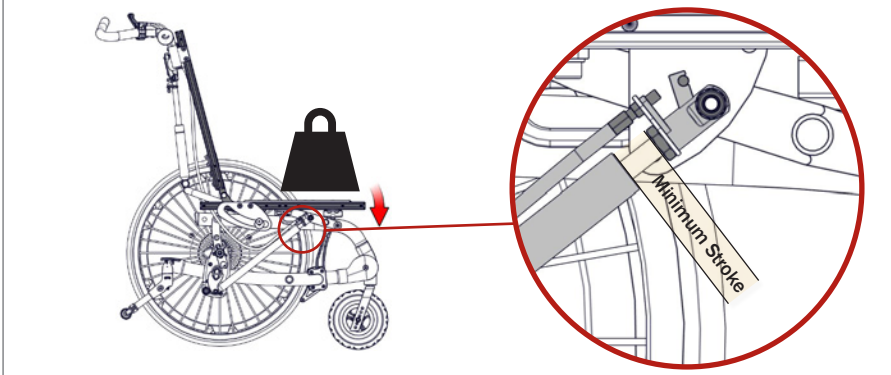
4.

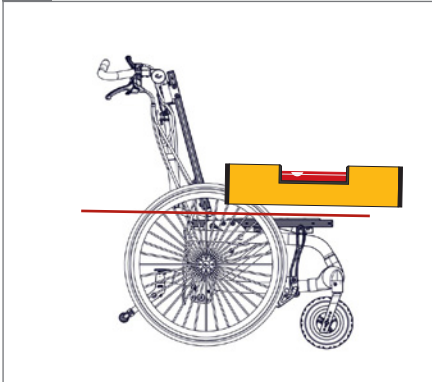
5.


6.

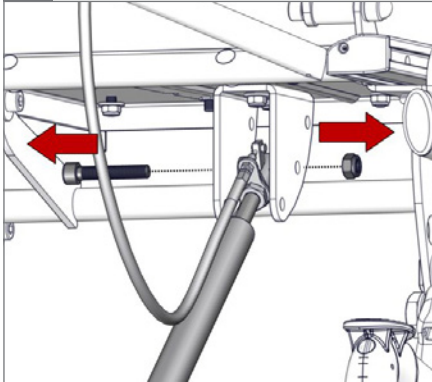⊙ 6 mm & ⚙ 13 mm

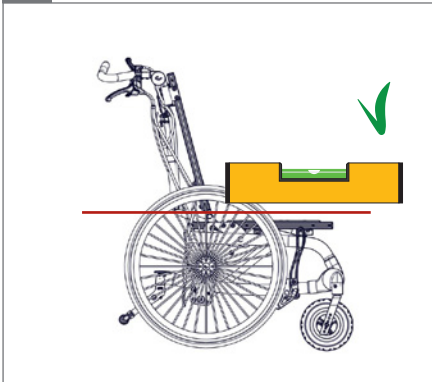
7.

8.

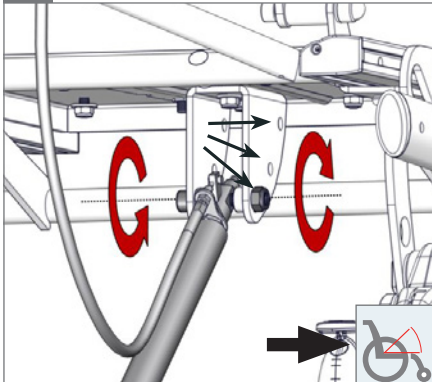
⊙ 6 mm & ⚙ 13 mm



02 

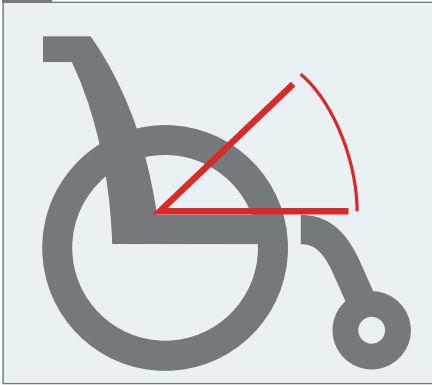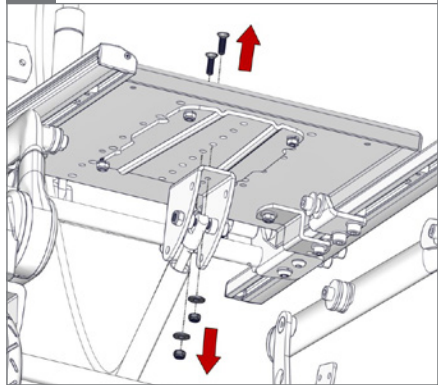1.  $\varnothing$  4 mm &  $\varnothing$  10 mm

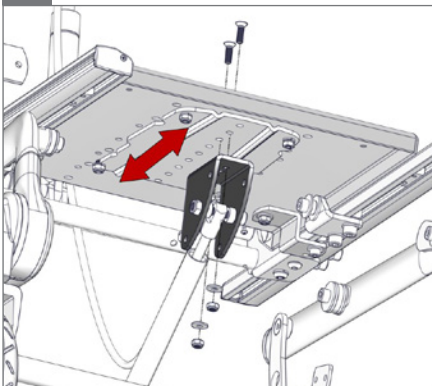
2.













3.

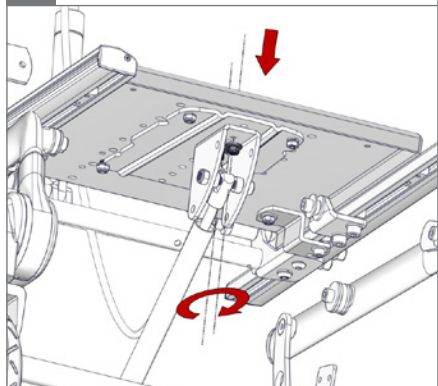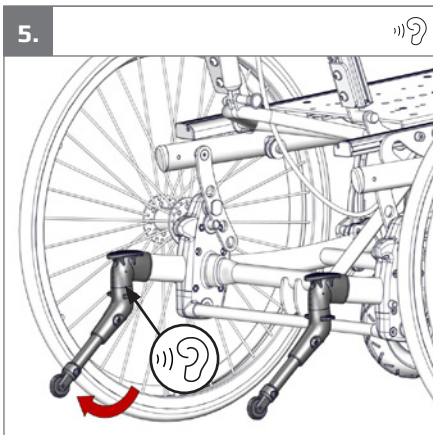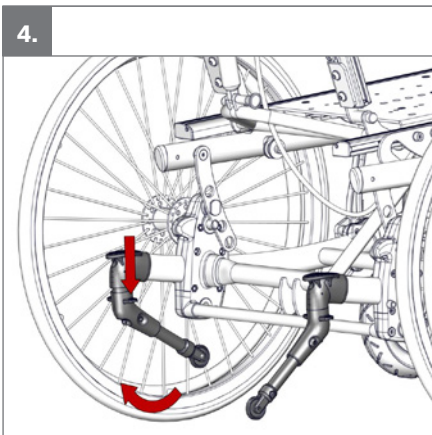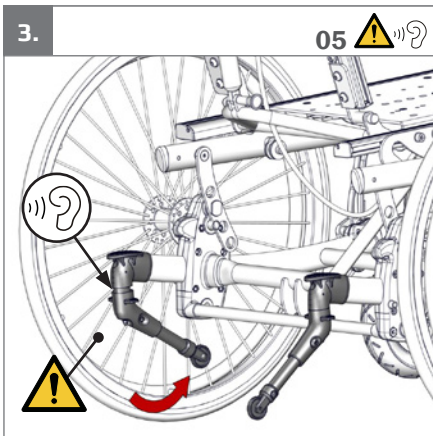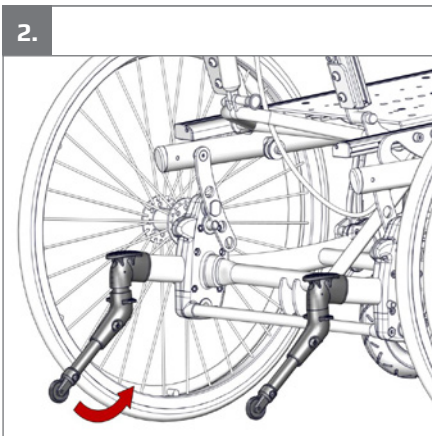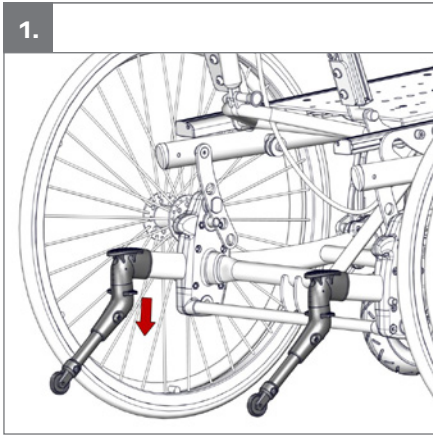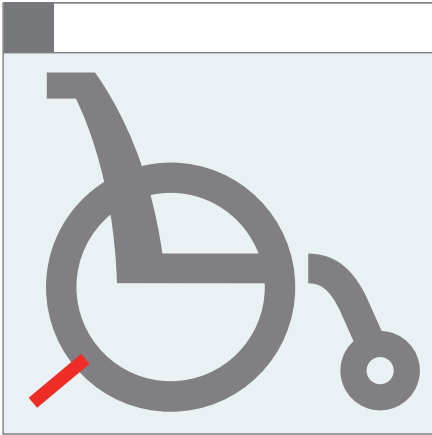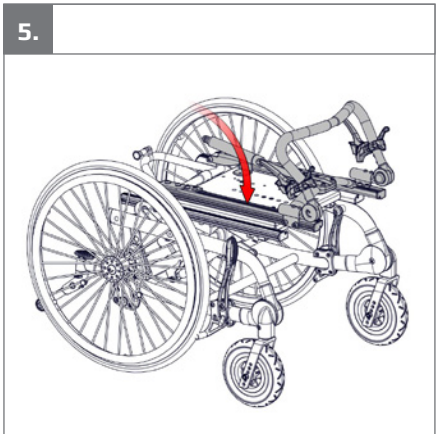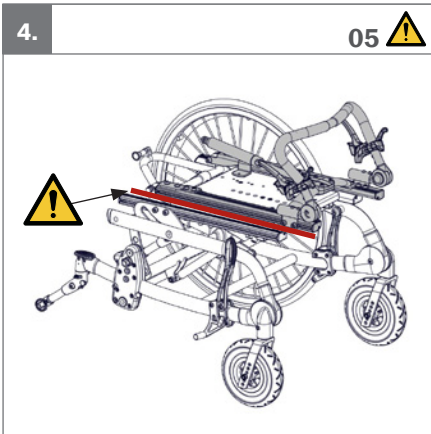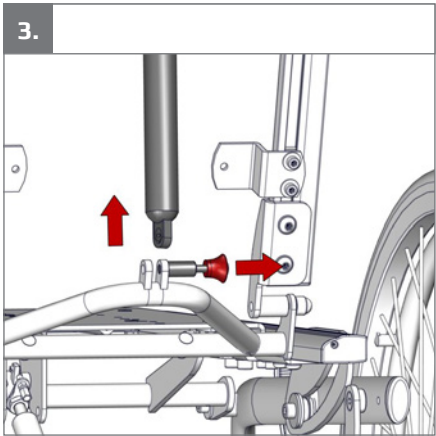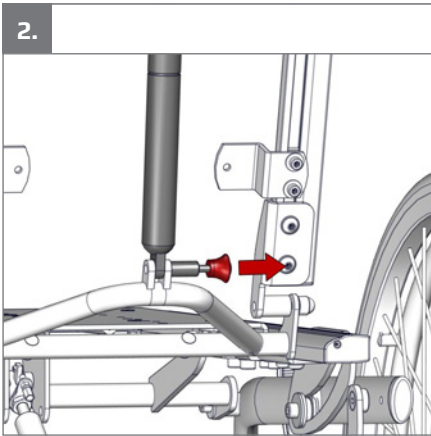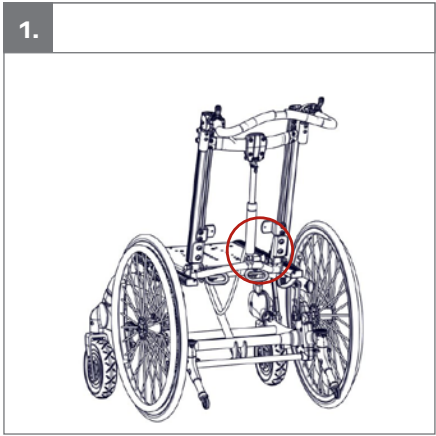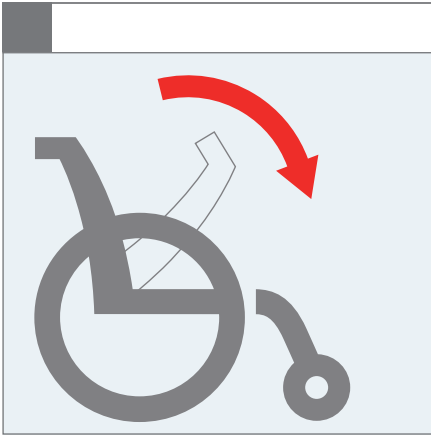$\varnothing$  4 mm &  $\varnothing$  10 mm

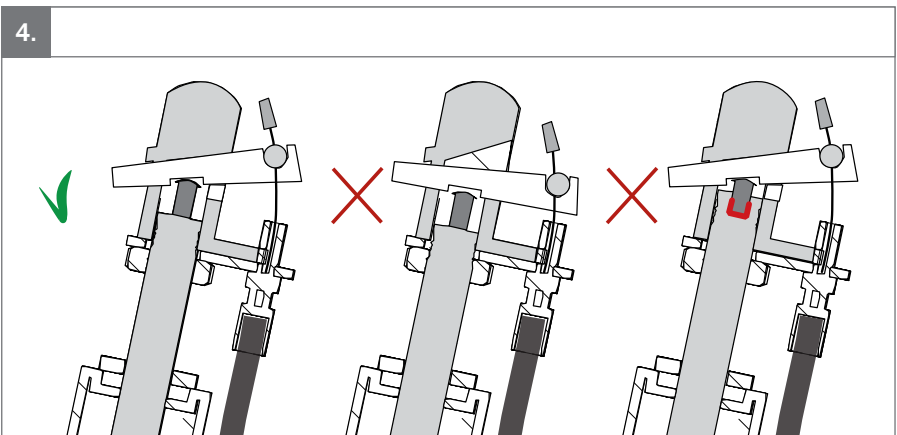
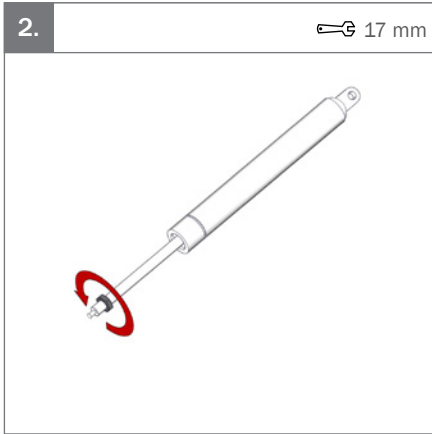
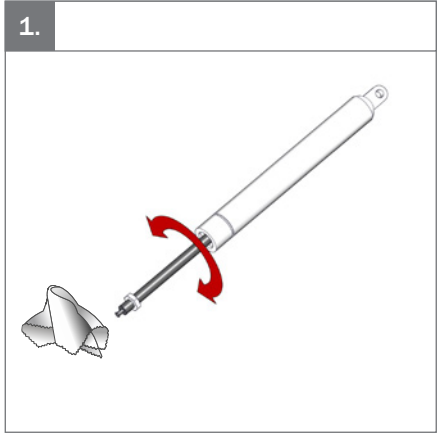
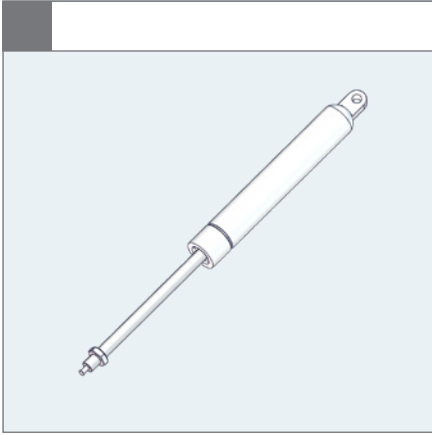
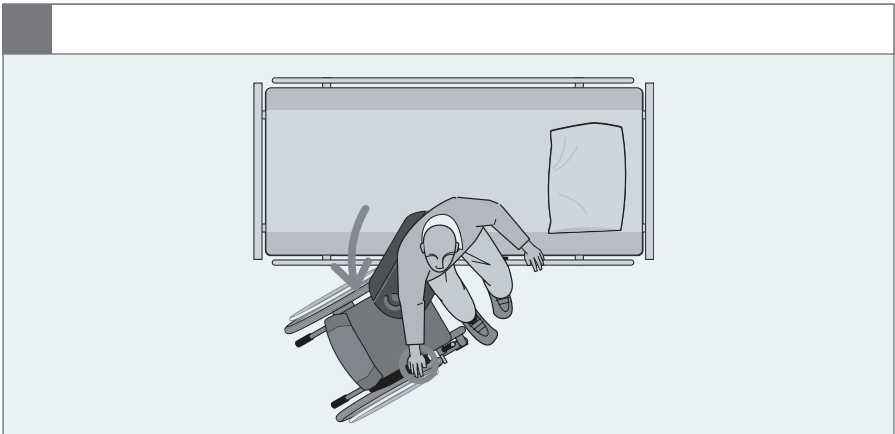
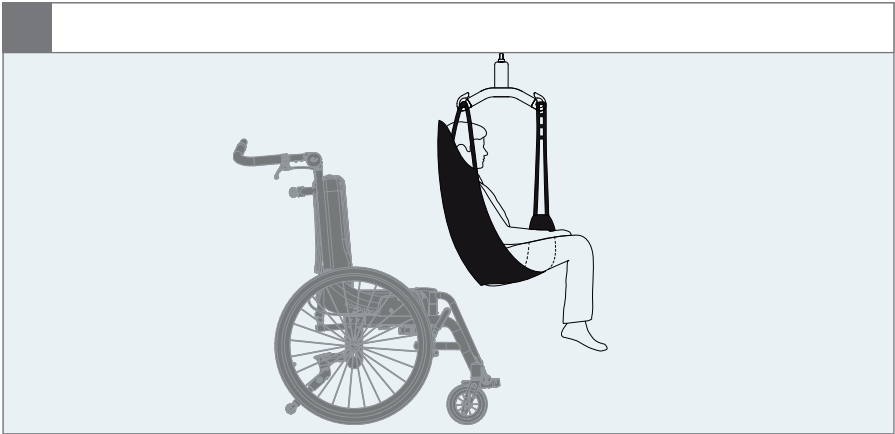
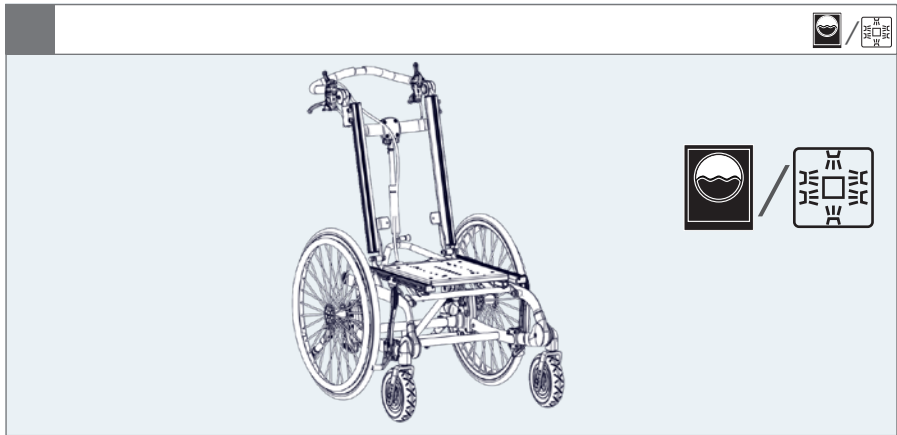
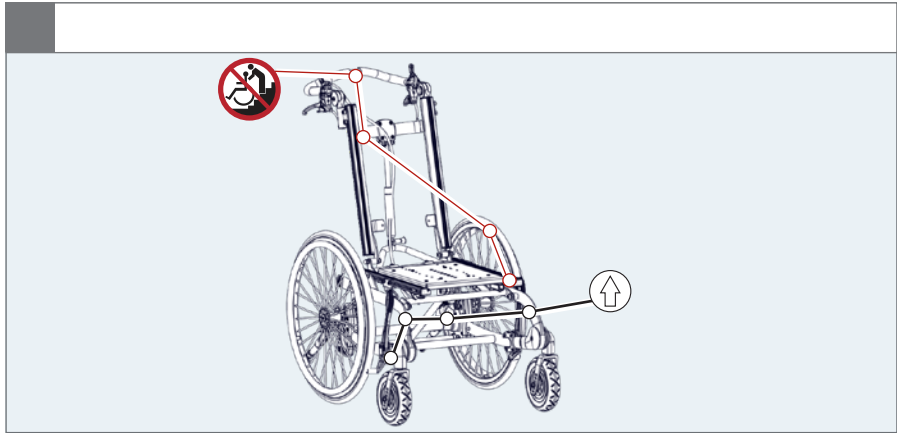
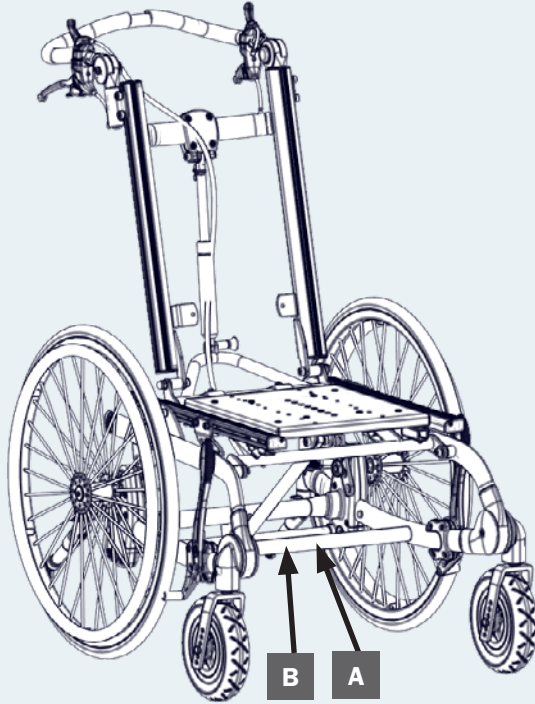
**A**

**R82**  
 by Etac

R82 A/S, Parallevej 3  
 DK-8751 Gedved  
 etac.com  
 Made in xxxxxxx

MD

CE

UK

YYYY-MM-DD

**B**

Posting group

Product name, size x

DMR ver.: xxxx

SN: xxxxxxxxxxxxxx

REF: xxxxxxxxxxxxxxxxxxxx

▲ Max. load: xxx kg  
 ▲ Max. load: xxx kg  
 ▲ Max. rated load: xxx kg  
 ▲ Mass device: xxx kg

Barcode: (00)xxxxxxxxxxxxxxxxxx(00)xxxxxx(xx)xxxxxx

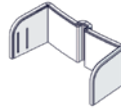
**mm  
(inch)  
kg  
(lb)**

<b>mm (inch) kg (lb)</b>	 1	 2	 3	 4
 Overall width	580 mm (22¾")	630 mm (24¾")	680 mm (26¾")	730 mm (28¾")
 Width between wheels	440 mm (17¼")	490 mm (19¼")	540 mm (21¼")	610 mm (24")
 Overall Length	830 mm (32¾")	830 mm (32¾")	830 mm (32¾")	830 mm (32¾")
 Overall height	900 mm (35½")	940 mm (37")	1040 mm (41")	1140 mm (45")
 Back height	430 mm (17")	470 mm (18½")	570 mm (22½")	670 mm (26½")
 Base plate (width x depth)	220×250 mm (8¾×10")	270×300 mm (10×11¾")	320×350 mm (12½×13¾")	370×400 mm (14½×15¾")
 Folded height	550 mm (21¾")	550 mm (21¾")	550 mm (21¾")	550 mm (21¾")
 Folded length	800 mm (31½")	800 mm (31½")	840 mm (33")	940 mm (37")
 Seat surface at front edge	430/460/490 mm (17/18/19¼")	430/460/490 mm (17/18/19¼")	430/460/490 mm (17/18/19¼")	430/460/490 mm (17/18/19¼")
 Footrest to adaptorplate	100–250 mm (4–9¾")	100–250 mm (4–9¾")	180–330 mm (7–13")	180–330 mm (7–13")
 Seat plane angle	0° - 40° (5° - 45°)	0° - 40° (5° - 45°)	0° - 40° (5° - 45°)	0° - 40° (5° - 45°)
 Back angle	0° - 30°	0° - 30°	0° - 30°	0° - 30°
 Minimum turning radius	600 mm (23½")	600 mm (23½")	600 mm (23½")	600 mm (23½")
 Maximum load	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)
 Maximum load at transportation	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)
 Mass device	22 kg (48.5 lb)	23 kg (50.5 lb)	24 kg (53 lb)	27 kg (59.5 lb)
 Mass device without rear wheels	19 kg (41.8 lb)	20 kg (44 lb)	21 kg (46.2 lb)	24 kg (52.8 lb)
 Maximum rated load	124 kg (273.3 lb)	125 kg (275.5 lb)	126 kg (277.7 lb)	129 kg (284.3 lb)
 Static stability downhill, uphill and sideways	9°	9°	9°	9°

9996097152

9996097338

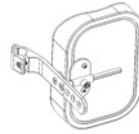
9996097153


9996097339

9996097134  
9996097129  
9996097128


9996097145

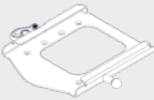9996097714

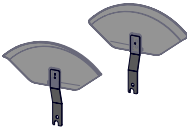9996097166

9996097148  
9996097150

9996097168

9996097737

9996097167

9996097114

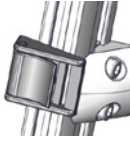
9996097169

9996097755


9996097221

9996097600

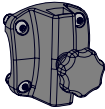9996097165

9996097151

**EN:** Warning: This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger  
 01 = Entrapment Hazards  
 02 = Tilting Hazards  
 03 = Transportation in motor vehicles  
 05 = Squeezing Hazard

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## General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and adults with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

### Intended purpose

The device is a frame for seating systems for children and adults with disabilities.

### Indications

The device is used together with a seating system (i.e. a modular seating system like R82 x:panda or R82 Panda Futura). It can be used by children and adults with disabilities, who lack the ability to maintain a seated position and move around (Within the room or between rooms / outdoors) without an assistive device. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age.

The user is the person sitting in the seat used with the frame. The caregiver is the person operating the frame. Never leave the user unattended in this product. Ensure permanent supervision by a caregiver. The use of the device will be discontinued more times during the day where the user will be lying down, standing or sitting in another assistive device or a conventional product.

For transportation the device is suitable for children and adults in the weight range of 22 kg to specified max user weight for transportation.

### Contra indications

There are no known contra indications

### Precautions

There are no known precautions

### Application areas

The product is for indoor and outdoor use

### Enclosed tools:

- 3, 4, 5 and 6 mm Allen key

## CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12183

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other

products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Alignment of user weight/mass of the combined seat and base, approval for transportation in vehicles of both products (ISO16840-4 for seat and ISO7176-19 for base)
- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

### Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

### Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

### R82 Warranty

[www.etac.com/support](http://www.etac.com/support)

### Presales information and instructions

Presales information and instructions are available at [www.etac.com](http://www.etac.com), under your product, in the documents section.

## Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions. Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

### Caregiver

- Read the user manual carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/ accessories.
- Always use the product in a moderate pace. Do not run with the product
- Information about transportation can be found in the document "M1460 Transportation in Motor Vehicles" supplied with the product
- Information about roadability can be found in the document "Roadability" supplied with the product
- Information regarding frame and seat combinations including max. load in combination can be found at: [etac.com](http://etac.com)

### Environment

- Observe the surface temperature on the product, before placing a user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to

measurements

### User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

### Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Removable parts without use of tool: wheels and foot support. The removal will have adverse effect on the device
- Activate the brakes before placing the user into the product
- Activate the anti-tips when the product is being used on inclines
- Activate the anti-tips before angling the seat and back
- Deactivate the anti-tips when forcing kerbs higher than 100 mm
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Ensure the wheels are fully-functional and secure before use
- The correct tyre pressure is printed on the tyre casing and listed in the section for tyre technical data.
- In case of a tyre puncture do not use the product until remedied
- Unoccupied device is suitable for land and air transport
- Attaching additional loads can adversely affect stability. Position of points intended to carry additional loads are specified in the relevant Mounting Guide referred to under section for accessories
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring

- 
- The product is approved for transportation in vehicles according to ISO 7176-19. More information can be found in the document "M1460 Transportation in Motor Vehicles" supplied with the product.
  - In accordance with ISO 7176-19 Annex D, the product is evaluated regarding the ability to accommodate vehicle-anchored pelvic and shoulder belts. The device receives an overall score of "Good"

## Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

### Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

### Reuse

This product and its accessories/components are suitable for reuse.

### Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

### Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

### Service checklist:

Check and adjust the following as per their intended use:

- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- If a seat adapter is mounted, check that the red handle for locking works and that the safety lock "clicks into position when placing a seat into the adapter
- Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required
- Quick release wheels are removable/replaceable
- Tire pressure
- Before storage: Always deactivate wheel locks/brakes
- Anti-tips can be fixed in working and resting position
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

Repair or replace any damaged or worn out parts.

### Maintenance

Before every use it is recommended to: Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use carry out a visual inspection, for any damaged or worn-out parts.

### Washing

#### Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

#### Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

### Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

### Materials

- Aluminium
- Rubber
- Plastic
- Stainless steel
- Steel

### Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

## Safety product

Due to variant and settings the wheelchair may become larger than the recommended dimensions. Compliance with recommended overall width and overall length ensures unhindered use of emergency escape routes. (See table 1)

**Table 1**

Recommended dimensions	Ready for use (mm)
Overall length – maximum	1200
Overall width – maximum	700
Overall height – maximum	1200
Turning diameter – maximum	1000
Pivot width – maximum	1300
Ground clearance - minimum	30

## Product safety

Due to variant and settings the wheelchair may not meet the limits for wheelchairs transportable by train. Measures can be taken to verify whether the specific product in question meets the limits. (See table 2)

**Table 2**

Wheelchairs transportable by train - limits	
Overall length – maximum	1200 mm + 50 mm for feet
Overall width – maximum	700 mm + 50 mm on each side for hands
Overall height – maximum	1375 mm (including 95th percentile male occupant)
Turning diameter – maximum	1500 mm
Smallest wheels	Accommodate a gap of 75 mm horizontal and 50 mm vertical
Weight – maximum	200 kg for manual wheelchair and occupant (including baggage)
Obstacle height that can be overcome – maximum	50 mm
Ground clearance - minimum	60 mm under foot rest with upward slope angle of 10° on top for going forward
Safe slope on which the wheelchair remain stable - maximum	6° dynamic stability in all directions 9° static stability in all directions, including with brake applied

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**Etac Ltd.**
**Gelbart AG**